


**Manual Escape-Mattress®
Mortipod**

1	Remove the Escape-Mattress® Mortipod from the storage bag and place it on the ground.	
2	Unfold the Escape-Mattress® Mortipod while unloosing the safety belt which holds it together. Open the rest of the safety belts when unfolded and open the zipper of the body bag.	
3	Place the body into the body bag and make sure the head is on the head rest.	
4	Close the zipper and fasten the 3 safety belts around the body-bag.	
5	Release the, in length adjustable, towing belts from the head and foot end of the Escape-Mattress® Mortipod. NOTE: The towing straps are equipped with extra straps for better and more ergonomic handling.	

6	<p>The deceased can now be moved by using the towing straps on the foot end and head end. Escape Mobility Company advises to always perform this with 2 people (towing) or 4 people (carrying).</p>	
7	<p>When the body is out of the mattress, the top body bag of the Escape-Mattress® Mortipod can be detached from the bottom of the mattress by unzipping both ends. This allows easy cleaning of the inside of the body bag.</p> <p>NOTE: the top body bag is also available as an additional article (code 21052).</p>	
8	<p>Always check the Escape-Mattress® Mortipod before storing for any damage.</p>	
8 bis 1	<p>If necessary clean, and disinfect, with water and disinfectant soap.</p>	
9	<p>Place the Escape-Mattress® Mortipod back into its designated place.</p>	

Attention:

The Escape-Mattress® Mortipod must be used by trained persons only.
 Training is only allowed to be given by persons who have been certified by Escape Mobility Company.
 Do not throw away this user manual.