








**Manual Escape-Mattress®
Economy Line / Escape-Pad**

1	Remove the Escape-Mattress® Economy Line / Escape-Pad from the storage bag and place it on the ground.	
2	Unfold the Escape-Mattress® Economy Line / Escape-Pad while unloosening the safety belt which holds it together. Open the other safety belt when unfolded.	
2 bis 1	Place the adaptable head rest on the Velcro straps at the head end (Economy Line).	
3	Place the evacuee on the Escape-Mattress® Economy Line / Escape-Pad. Make sure the head is on the adaptable head rest (Economy Line - picture left) or on the wide green strap side (Escape-Pad - picture right). NOTE: a pillow can be put under the green strap (Escape-Pad) for extra comfort.	
4	Place the feet of the evacuee in the foot bag (Economy Line - picture left). Or strap the lower legs by unloosening the buckles at the foot end, with the foot strap (Escape-Pad - picture right) and close the buckles again when strapped.	

5	<p>Close the 2 safety belts over the chest and the upper legs of the evacuee.</p> <p>NOTE: Make sure the arms of the evacuee don't stick out of the mattress.</p>	
6	<p>The evacuee can now be moved to safety using the towing straps on the foot end and head end. Escape Mobility Company advises to always perform an evacuation using the Escape-Mattress® Economy Line / Escape-Pad with 2 people.</p> <p>NOTE: the Escape-Mattress® Economy Line / Escape-Pad can only be used as a towing mattress. Max towing capacity is 150kg.</p>	
7	<p>Unfasten the safety straps and help the evacuee get off the Escape-Mattress® Economy Line / Escape-Pad.</p>	
8	<p>Always check the Escape-Mattress® Economy Line / Escape-Pad before storing for any damage.</p>	
8 bis 1	<p>If necessary clean, and disinfect, with water and disinfectant soap.</p>	
9	<p>Place the Escape-Mattress® Economy Line / Escape-Pad back into its designated place.</p>	

Attention: The Escape-Mattress® Economy Line / Escape-Pad must be used by trained persons only.
 Don't use a damaged mattress anymore.
 No alternations may be made to the Escape-Sheet, except by Escape Mobility Company.
 Do not throw away this user manual.